

Surgistar, Inc.
MFG of Single-Use Ophthalmic, ENT,
and Other Surgical Cutting Instruments
ISO 13485 and CE certified

Surgical Eye Shield (Nonsterile)
Instructions for Use

WARNINGS

- The user must be fully acquainted with surgical procedures and technique.
- Do not use the eye shield if its packaging is damaged or if there is evidence of any other abnormalities.

CONTRAINDICATIONS AND PROHIBITIONS

- Do not use for any purpose that is not indicated by the treating ophthalmologist.

SHAPE AND STRUCTURE

1. Main materials: Polycarbonate

INTENDED USE

Indications: Surgistar eye shields are used in treatment procedures for various diseases to protect the patient's eye(s). As they are non-sterile products, they are not recommended for use in an open surgical area. As such, they should only be used by trained and licensed physicians. The specific nature of the procedure will determine the physician's choice of product and techniques to be used.

PERFORMANCE, PURPOSE AND EFFECT

To protect the eye(s) of the patient

OPERATION AND USE

- Use in accordance with instruction.

CAUTION

- Use the eye shield as soon as it is unpacked.
- Do not use deformed or fractured eye shield
- Do not use excessive force or use with inappropriate equipment.
- To prevent infection, dispose of with care in a safe and environmentally friendly manner, such as in a medical waste basket.

CONDITIONS OF STORAGE:

Surgistar eye shield should be stored at ambient temperature away from possible contamination from biohazards and water damage.

INSTRUCTIONS

1. In procedures where there exists the need to protect the eye(s) of the patient, the physician will select the size and style of Eye Shield indicated for the patient and procedure to be performed.
2. Remove product from packaging carefully. Care should be taken when removing eye shields from packaging, in handling, and in disposal.
3. Physician affixes the Eye Shield to cover one or both eyes using surgical tape. Perform procedure. After the procedure and hazards to the eye have been eliminated, remove the Eye Shield and Surgical Tape from the patient.
4. After use, discard the Eye Shield by an appropriate method to avoid contamination to other personnel who may come into contact with the Eye Shield.



Surgistar, Inc.
2310 La Mirada Drive
Vista, California 92081 USA
Telephone: (760) 598-2480
Fax: (760) 598-2481
Email: customerservice@surgistar.com
www.surgistar.com